

# Lightwriter SL40 Series in-use bag

## Fitting instructions



- 1** Make sure that your Lightwriter is turned off - from the main menu go to Setups (option 7), then Power (option 7) then Power Off (option 2), then press Do
- 2** Turn your Lightwriter over on a table or flat surface, so that the keyboard is face down and the screens are facing away from you. Take note of the 4 gold coloured screw holes.
- 3** Unzip the main compartment of your in-use bag and turn it upside down with the flap furthest away from you – the logo should be the right way up.
- 4** Place the bag on top of the Lightwriter and line up the 4 eyelets with the gold screw holes.
- 5** Loosely screw in the 4 plastic screws by hand and make sure everything is in place before tightening them with a screwdriver. Do not over-tighten as you may damage them.
- 6** Turn over your Lightwriter, zip the flap up and you are ready to go.



For more help and the latest information please visit: [www.abilia.uk](http://www.abilia.uk)



ABILIA