



Safer everyday life

Emfit SafeBed

Emfit SafeBed is a "missing person alarm" that can register when a person gets out of bed, or fails to return to bed within a pre-set time.

This provides increased security for you if you have a tendency to rise up and walk away unattended as the alarm draws attention of personnel or relatives who are in the vicinity.

Art.nr: 464001



ABILIA

SAFEBED

Emfit SafeBed consists of a sensor that is placed under the mattress, and a main unit where settings are entered.

If the time setting is short (e.g. 3 seconds) the alarm will sound when the person gets out of bed. By increasing the time (up to 30 minutes) the alarm will sound if the person does not return to bed within the pre-set time.

The sensor sends a signal to the control unit which sounds an adjustable alarm that can be used to alert nearby nursing staff. It is also possible for the control unit to send the alarm further such as to a pager or a cellphone, using our accessories All in One.

Emfit Safebed is part of a series of reliable alarm products which are well tested and used both at home and nursing homes around the world.



The sensor is placed under the mattress

TECHNICAL INFORMATION					
SENSOR		CONTROL DEVICE		ART.NO	ACCESSORIES
Size	400 x 580 x 1,5 mm	Size	97 x 127 x 30 mm	464015	Power supply
Material	Polyester	Weight	150 g	464005	Wall mount
Cabel lenght	300 mm	Color	White	330701	All in One*
		Battery	2pcs 1,5 V, AA-battery	330700	All in One Basic*
* Not available at all markets					