



Control both your physical
and virtual world

GEWA Connect

GEWA Connect helps you control your physical world, such as your TV, lights, windows and doors, or z-wave units such as Andromeda Socket.

It will also help you to stay in contact with family and friends by making phone calls and sending text messages. You can stay up-to-date by using Facebook and other social media or by reading the latest news online, watching YouTube or playing music by controlling your smartphone with GEWA Connect.

Art.nr 461188



ABILIA

CONTROL YOUR PHYSICAL AND VIRTUAL WORLD


With GEWA Connect the physical and virtual world are merged and can both easily be controlled. Just install GEWA Connect on your own device and start to control your home and virtual world from your smartphone or tablet.

HOW DOES GEWA CONNECT WORK?


The heart of the system is the Abilia Multibox that is connected to your device via Bluetooth. The GEWA Connect app is installed on your device and receives commands from the switches attached to the Multibox. Remote commands are sent from the app to the IR or GEWA Radio transmitter on the Multibox to control your environment.



GEWA Connect point scanning on a tablet

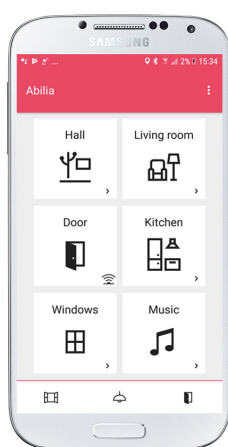

Grid view

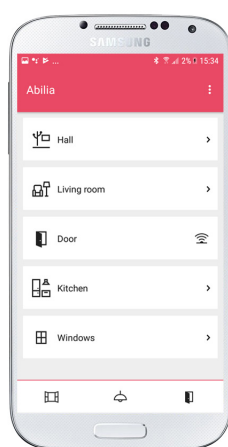

List view

Based on your own needs, you can choose to manage your environment with either List view or Grid view.

The system is operated by button switches, sip and puff controls or any type of switch that suits your needs.

You can easily alternate between row and point scanning depending on which app and function you are using in that moment.

AN ALERT FUNCTION TO KEEP YOU SAFE

To keep you safe, you will find a built-in alert function which lets you call for help, any time you need assistance. Just press the switch for 8 seconds to send an alert signal to a predefined receiver. The function is performed in the Multibox, which means that if the device is unavailable for any reason, you can still send the alert.

SUPPORTS YOU ON THE GO

Take GEWA Connect with you and stay in control when you are on the go. You can teach the system remote control commands for places you regularly visit outside of your home. The ability to control the virtual world is of course not restricted to your home.

SIMPLE AND SAFE MANAGEMENT

Thanks to myAbilia - our online service, you can manage all settings and configurations from a remote computer and everything is stored in a safe place. This means that if you lose your phone or tablet for some reason, your configuration and data are easy to download to your new device.

PRODUCT
GEWA Connect

ART NR.
461180

TECHNICAL INFORMATION
Switch Input 3.5 mm jack
Transmit methods IR and GEWA Radio
Weight 153 g
Size: (HxWxD) 108 x 67 x 22

DELIVERED WITH
- Multibox incl. charger
- Manual
- Licence voucher
- Device recommendations