



Safer everyday life

Emfit SafeBed

Emfit SafeBed is a "missing person alarm" that can register when a person gets out of bed, or fails to return to bed within a pre-set time.

This provides increased security for you if you have a tendency to get up and walk around unattended, because the alarm draws attention to assistants or relatives who are nearby.

Art.nr: 464001



ABILIA

SAFE BED

Emfit SafeBed consists of a sensor that is placed under the mattress, and a main unit where settings are entered.

If the time setting is short (ie. 3 seconds) the alarm will sound when the person gets out of bed. By increasing the time (up to 30 minutes) the alarm will sound if the person does not return to bed within the pre-set time.

The sensor sends a signal to the control unit which sounds an adjustable alarm that can be used to alert nearby nursing staff. It is also possible for the control unit to forward to a further alarm, such as pager or mobile phone, using our All in One accessories.

Emfit epilepsy alarms are part of a series of reliable alarm products which are well tested and used in both home and residential care around the world.



The sensor is placed under the mattress

TECHNICAL INFORMATION

SENSOR

Size 400x580x1.5mm
Material Polyester
Cable length 300mm

CONTROL DEVICE

Size 97x127x30mm
Weight 150 g
Colour White
Battery 2x 1.5V, AA batteries

ART.NO ACCESSORIES

464015 Power supply
464005 Wall mount
330701 All in One*
330700 All in One Basic*

* Not available at all markets