



Control both your physical
and virtual world


GEWA Connect

GEWA Connect helps you control your physical world, like TV, lights, windows and doors or z-wave units such as Andromeda Socket.

It will also help you to stay in contact with family and friends by making phone calls and sending text messages. You can stay up-to-date by using Facebook and other social media or by reading the latest news online, watch YouTube or play music by controlling your smartphone with GEWA Connect.

This makes you become more independent and feel safe both in your home and at work.

Art.nr 461180

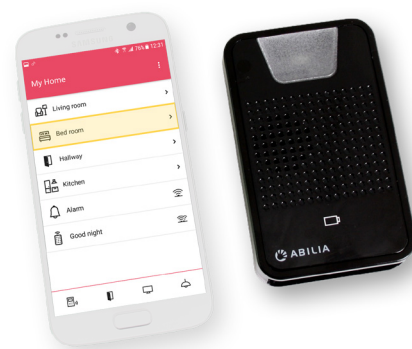

CONTROL YOUR PHYSICAL AND VIRTUAL WORLD


With GEWA Connect the physical and virtual world are merged and both can easily be controlled. Just install GEWA Connect on your own device and start to control your home and virtual world from your smartphone or tablet.

HOW DOES GEWA CONNECT WORK?


The heart of the system is the Abilia Multibox that is connected to your device via Bluetooth. The GEWA Connect app is installed on your device and receives commands from the switches attached to the Multibox. Remote commands are sent from the app to the IR or GEWA Radio transmitter on the Multibox to control your environment.



GEWA Connect point scanning on a tablet

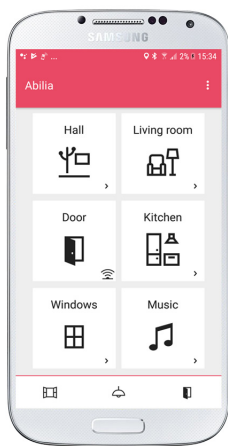

List view

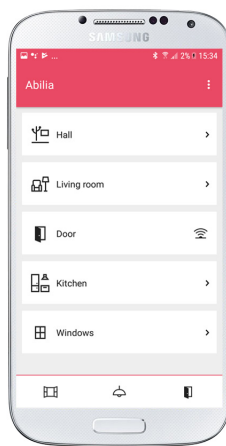

Grid view

Based on your own needs, you can choose to manage your environment with either list view or grid view.

The system is operated by button switches, sip and puff controls or any type of switches that suits your needs.

You can easily switch between row and point scanning depending on what app and what function you are using in that moment.

AN ALERT FUNCTION TO KEEP YOU SAFE

To keep you safe, you will find a built-in alert function which let you call for help at any time you need assistance. Just press the switch for 8 seconds to send an alert signal to a predefined receiver. The function is performed in the Multibox, which means that if the device for any reason is unavailable, you can still send the alert.

SUPPORTS YOU ON THE GO

Take GEWA Connect with you and stay in control when you are on the go. You can teach the system remote control commands for places you regularly visit outside of your home. The ability to control the virtual world is of course not restricted to your home.

SIMPLE AND SAFE MANAGEMENT

Thanks to our webservice myAbilia, you can handle all settings and configuration from a remote computer and everything is stored in a safe place. This means that if you lose your phone for some reason, your configuration and data are easy to download to your new phone or tablet.

PRODUCT	ART NR.	TECHNICAL INFORMATION		DELIVERED WITH
GEWA Connect	461180	Switch Input	3.5 mm jack	- Multibox incl. charger
GEWA Connect incl. phone	461190	Transmit methods	IR and GEWA Radio	- Manual
GEWA Connect incl. tablet	461195	Weight	153 g	- Licence voucher
		Size: (HxWxD)	108 x 67 x 22	- Device recommendations