

RELEASE NOTES FOR ROLLTALK

Release notes for Rolltalk Designer 6.8.0 release packages

2913: Office - Word editing with IntelliGaze

Solved with the introduction of IntelliGaze Desktop 2.0

1650: E-mail systems support

Support for popular mail providers, such as Gmail and Hotmail.

3713: IntelliGaze 2.0 page library

New page libraries for Intelligaze Desktop 2.0

3687: IG 2.0 support

Support for IntelliGaze Desktop 2.0

3404: Screen is wrong resolution after running some software (Pacman)

Screen resolution now re-set after running DOS programs or other software that uses custom resolutions.

3403: «Could not connect to server» after UCAN software has been installed

Solved "server is busy" message at startup on some configurations.

3328: Wheelchair driving macros

Prevents Rolltalk Designer editor to start sequences of digital ports from unknown state.

3339: Deleting calendar pages causes errors

Calendar pages can now be deleted without repercussions.

3430: Scanning with «Wait until sound is played» waits too long

Auditive scanning now has proper pause time between fields.

3606: IntelliGaze startup time is erroneous

Intelligaze registry files pointed to Designer instead of Workshop, now fixed for backwards compatibility.

3570: Dwell animation doesn't show on exit field on dynamic pages

Exit field on dynamic pages now has dwell animation.

3700: Email: Blind copy does not work

BCC of emails now allowed.

3649: Bug in year calendar

Fixed bug in Year calendar for 2012.

3802: “Dwell off” command is performed even if dwell is off

"Dwell off" now does not activate field commands when dwell is turned off.

3252: Date in calendar cannot be selected with IntelliGaze

Calendar dates can now be selected with IntelliGaze.

2789: Eye control in external games moves in set patterns

Blickfang is cleared when Windows mode is activated.

3413: New Facebook layout without “Enter” button breaks entering comments

Facebook comments fixed.

3826: Field highlight drawing issue

Fixed bug in drawing of field highlights

3840: “Coop billedesamlingen” contains black fields

Images can now contain "%" in filename.

3963: .avi and .wmv-files as attachments doesn't work

Videofiles can now be attached to emails and shown.

3898: IntelliGaze 2.0 integration

Rolltalk Designer will now be shown if there is no keyboard page set, and the keyboard page should be shown even if it is not adapted.

3900: Intelligaze demo for IG Desktop 2.0

IntelliGaze Desktop 2.0 demo included.

1252: Warning when starting another instance of Designer; «Another program is using the speech synthesis” followed by an exit.

Rolltalk Designer now defaults to opening only one instance at a time unless the “start program” command is used.